EN

bebird

Ear-care  
You Can See

Please consult the instruction before using this product

# App Download

or


Search for “bebird”  
in your phone’s app store.

Scan the QR code to download  
and install the bebird app

## App Connection

1 Press and hold the power  
button to turn on the device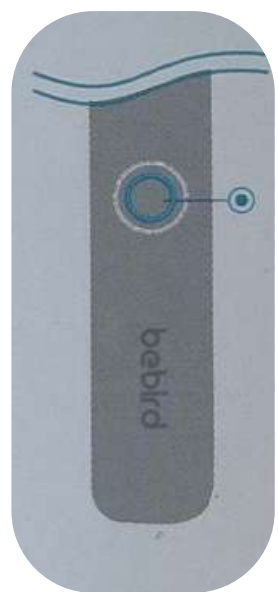


Once the device powers on,  
a blue light will illuminate.

2 Open the Bebird App, and  
click “Connect the device”


Connect the device

3 Connect Product Wi-Fi  
“bebird-xx-xxxxxx”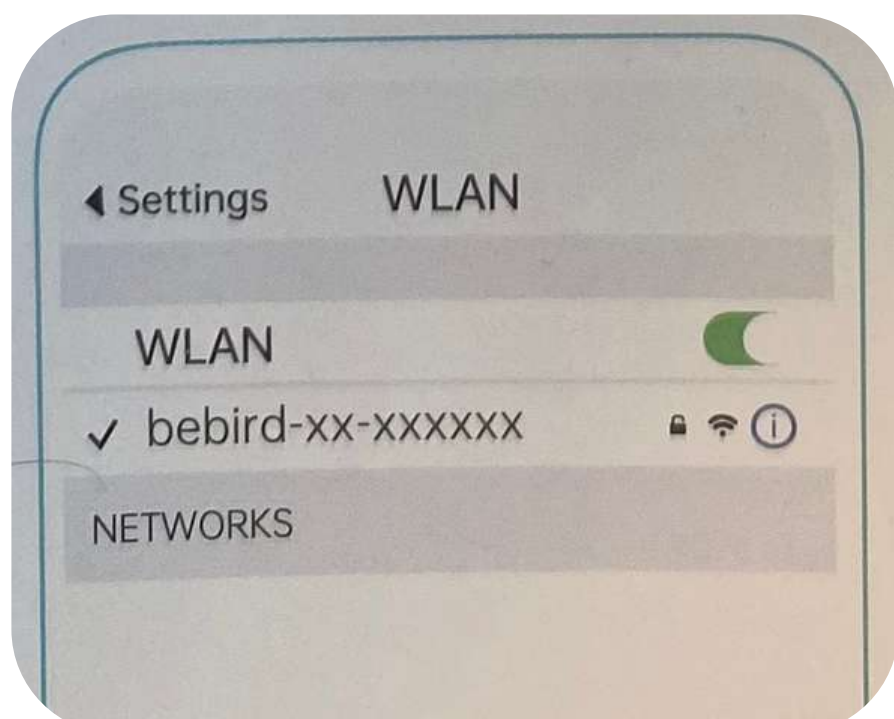

4 Return to the Bebird app and  
click “Begin to clean ears” to  
start using the device.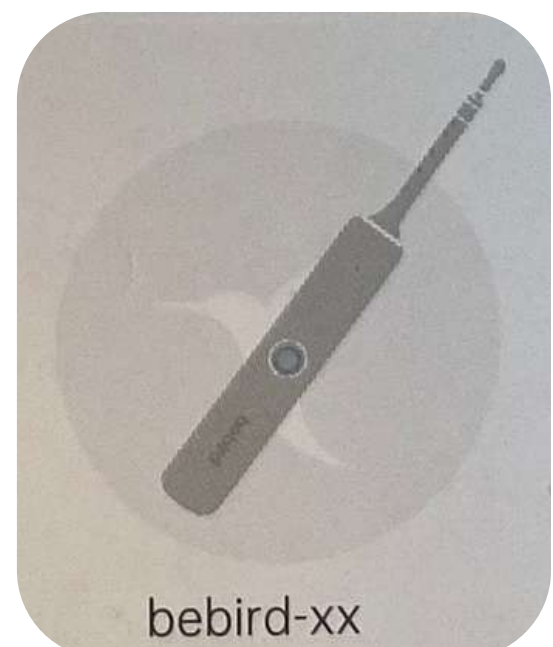

Begin to clean ears

# Device Overview

Silicone  
Covered  
Tips

Hose


Attention:

Please note that our hose is designed with a specific bending direction. When using, ensure the scale side is facing forward. The hose should only be bent left and right to avoid damage or reduced functionality

Do not bend in other directions, otherwise it may cause damage to the product.

Do not use brute force. Before using the tweezers and ENT set, the bent lens needs to be straightened again.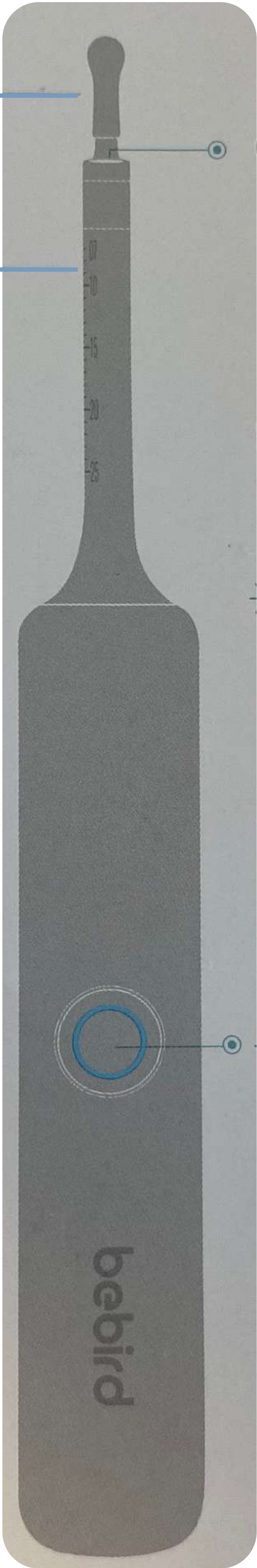

Camera Lens  
Bend the camera

Power Button


When the device is powered on, a blue light will illuminate.


During power-on and connection, the blue indicator light will flash.

Once connected, the blue light will remain lit.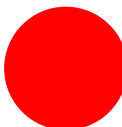


While charging, the indicator light will glow red.

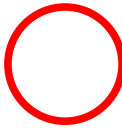When charging is complete, the red indicator will turn off.

The blue indicator will flash rapidly when the battery is low

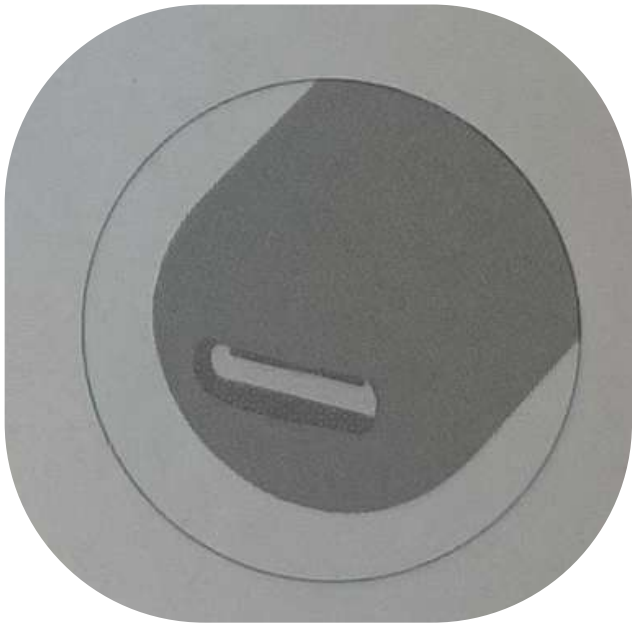Charging Port

# Accessory Instalation

## Protective Plug:

Place the ear canal protective plug in the appropriate position on the hose to prevent it from being inserted too deeply into the ear canal.

## Ear Spoon:

Unscre the current accessory by twisting it, counterclockwise to loosen and remove it.

Attach the desired accessory by screwing it in clockwise until its tight and secure

## Tweezers Components:


First, secure tweezers component A onto the main device, then twist on component B.

# How to use the tweezers components?

Place tweezers component-A onto the device body firstly, then install and tighten tweezers component-B


Gently pinch the handle of the tweezers with two fingers to close the tweezers.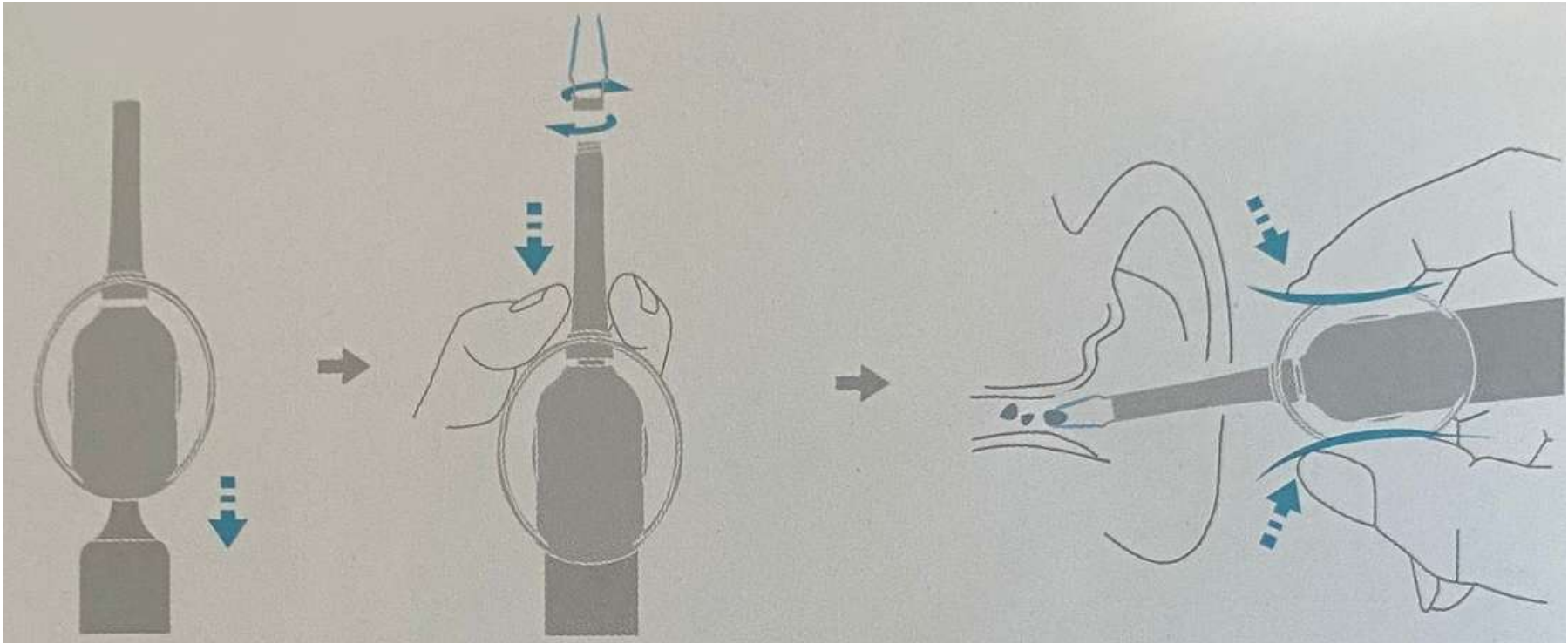

**Warm Reminder**

Install the tweezers component correctly before use to avoid its loosening or falling off

# How to use the Ear/Nasal/Throat Observation tools?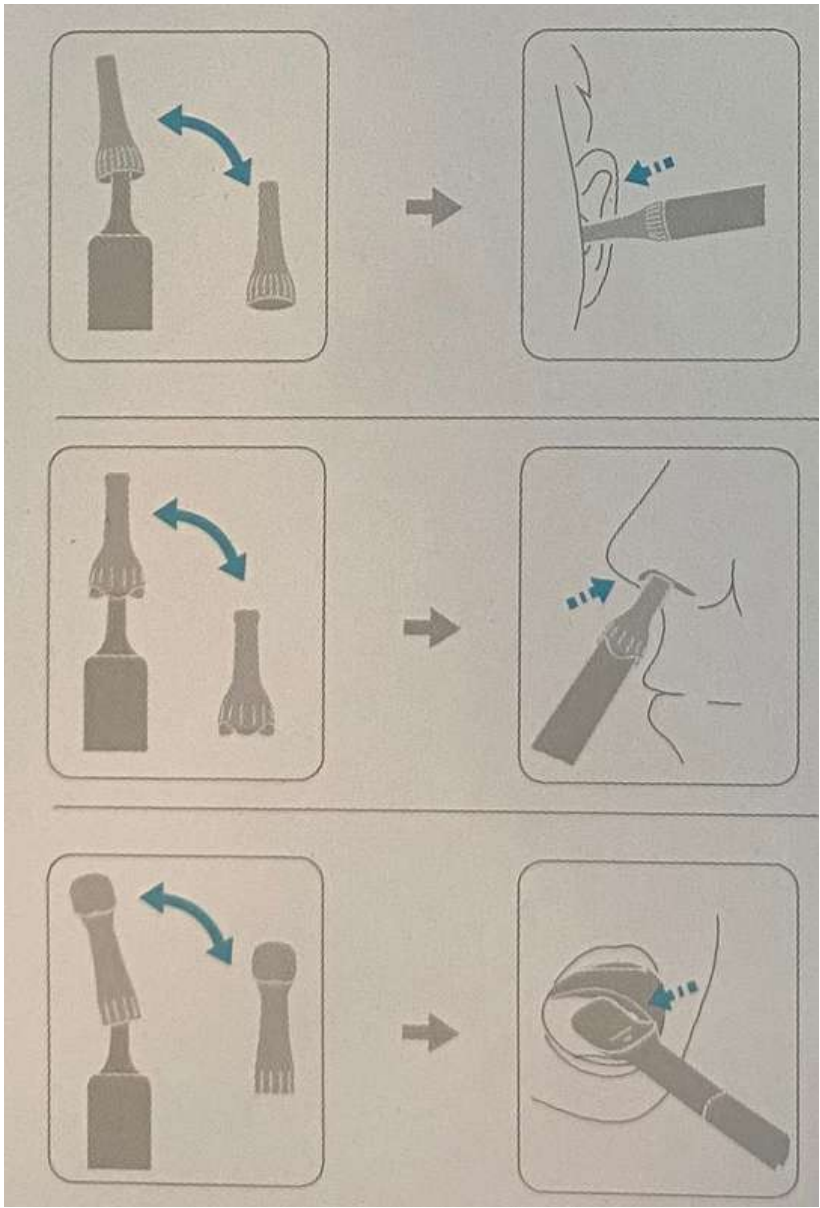

Put on the ear canal speculum tightly, and slowly go into the ear.


Put on the nasal speculum tightly, and slowly go into the nose.

Put on the tongue depressor tightly to prevent it from loosening.


**\*Note:**

Please make sure the tool is securely attached. If felt uncomfortable, please do not continue

# Accessory List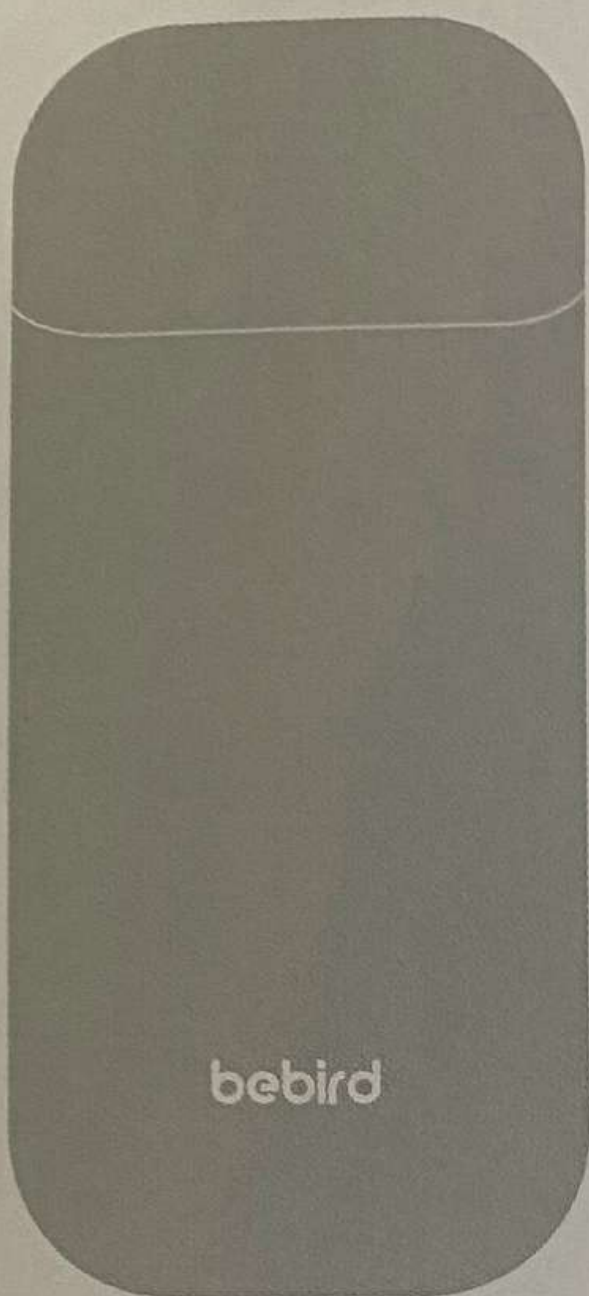

Accessory Case

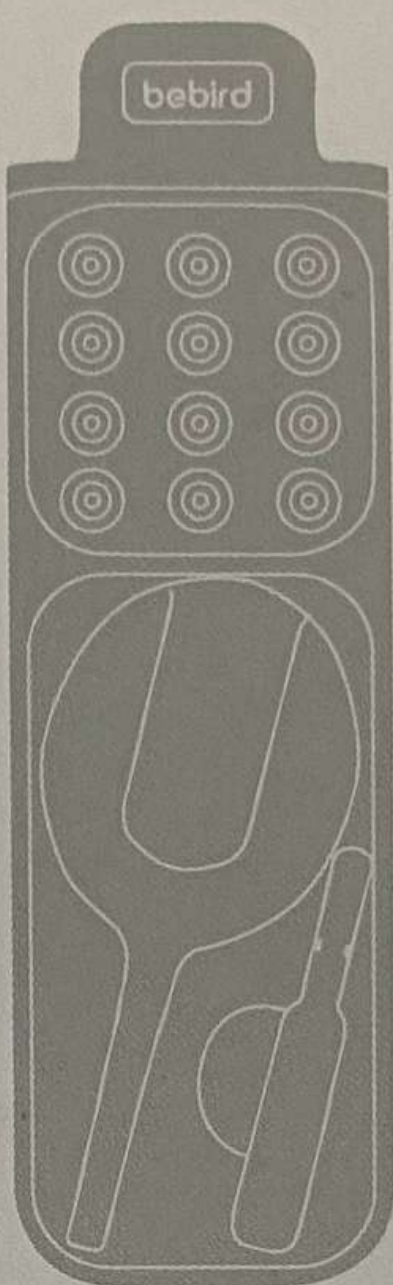Accessory Tray

Nasal Speculum


Ear Canal Speculum

Tongue Depressor

Metal, Silicone  
Wrapped Ear Spoon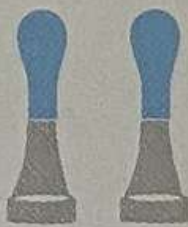

Double-color Q-  
elastic Ear Spoon

Double-color  
Annular Ear Spoon


Sponge Ear Spoon

Plastic Ear Spoon

Tweezers  
Component-B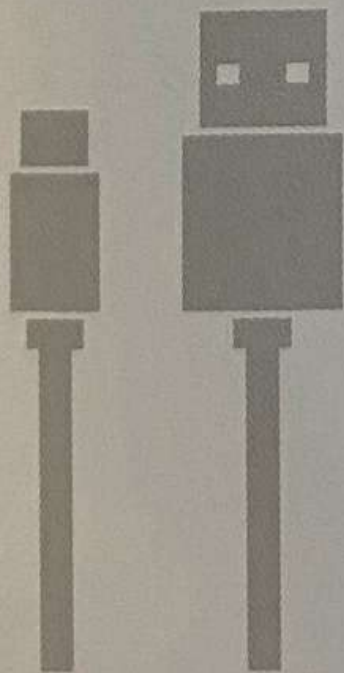

Charging Cable

Ear Canal  
Protective Plug

Lens Brush

Tweezers  
Component-A

# Troubleshooting

| Problem                                                               | Solution                                                                                 |
|-----------------------------------------------------------------------|------------------------------------------------------------------------------------------|
| The LED light of the camera is not on                                 | Use the enclosed charging cable to charge                                                |
| The app closes quickly                                                | Interrupt the connection, restart the device to connect again                            |
| The indicator light blinks quickly                                    | Delete the existing app and reinstall                                                    |
| Abnormal display in app                                               | Use the enclosed charging cable to charge                                                |
| The wifi connection was unstable and disconnected after a few seconds | Interrupt the connection, restart the device to connect again                            |
| Image is blurry                                                       | Wipe the camera lens with a double-ended cleaning brush, cleaning cloth, or alcohol swab |
| Movement feels unnatural                                              | Switch the left/right ear mode in the app interface                                      |

# Specifications

|                         |                                                                                          |
|-------------------------|------------------------------------------------------------------------------------------|
| Usage time              | Up to 80 minutes                                                                         |
| Charge time             | About 50 minutes                                                                         |
| Intelligent orientation | Gimbaled Intelligent Gyroscope                                                           |
| Image sensor            | CMOS                                                                                     |
| Optimum Focal Length    | 12.5mm                                                                                   |
| Input Current           | 5V=0.5A                                                                                  |
| Operating Frequency     | Wipe the camera lens with a double-ended cleaning brush, cleaning cloth, or alcohol swab |
| Lens grade              | Switch the left/right ear mode in the app interface                                      |
| Operating Environment   | 3P* Slim Lens, IP67 Waterproof, Dustproof & Antoi-Fog                                    |
| Image Frame Rate        | 20fps                                                                                    |

# Warnings

- Cleaning: Wipe the lens with a lens brush.
- Assembly: Ensure all device accessories are securely attached to prevent them from falling off during use.
- Operating Environment: Use the device in a suitable, private environment. Avoid using it in crowded spaces.
- Battery Maintenance: The device includes a built-in rechargeable battery. If unused for extended periods, charge it once a month to maintain battery health.
- Supervision for Children: Children should not use this device unsupervised to avoid accidental injury.
- Sunlight Exposure: Keep the main device and accessories out of direct sunlight to prevent softening or damage.
- Water Exposure: Do not submerge the device in any liquids.
- Temperature Notice: During use, the device may warm slightly due to the LED lights; this is normal and is capped at approximately 45°C (113°F).
- Intended Use for Tweezers: Use the tweezers only to remove earwax or other foreign objects from the ear canal.

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